



Jumbotag

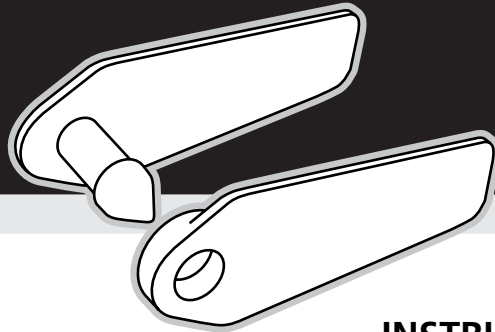


Step one: LOADING APPLICATOR

1. Pull each part from its packing strip and place both male and female sections of the tag into the correct jaws of the applicator. Ensure that the tag is perfectly lined up by gently closing the pliers until the point of the pin touches the female tag. The female tag should be positioned in the jaw with a hole.

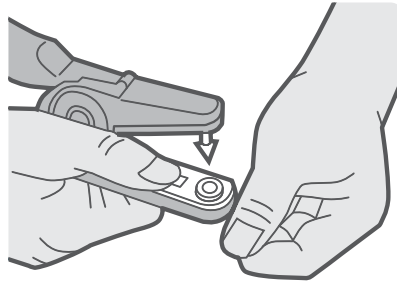


Step two: APPLICATION

2. Place thumb and forefinger close to the area the tag will pierce and apply the tag with a quick even pressure. The point should pierce the back of the ear, close to the thumb, leaving the point inside the ear.

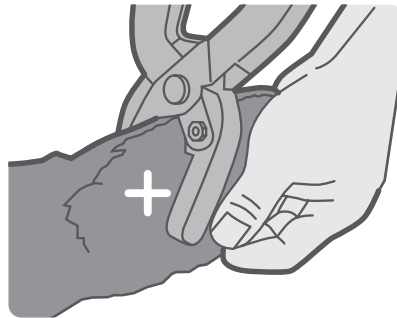


3. After the tag is fitted, check it is clear of the pliers before pulling away. It is then essential to quickly rotate the tag to allow any flesh that may be caught between the male pin and the female hole to drop out. The tag must be free fitting therefore avoiding any chance of infection or necrosis.


BEST TAGGING PRACTICE

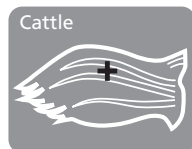


Cattle
Place in centre of ear
avoiding veins

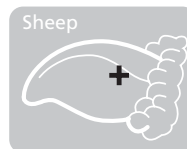

Sheep
Insert tag near
the middle of the ear
avoiding veins

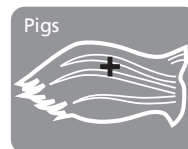

Pigs
Tag here avoiding
veins and gristle

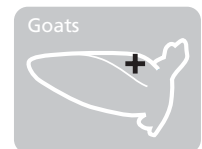

Goats
Insert tag near
the base of the ear
towards the top

CUSTOMER SERVICE

In the unlikely event that your applicator is damaged or tags have been damaged in transit, telephone our customer services dept. immediately. Where we shall do our utmost to solve the problem. Do not proceed with tagging under any circumstances!

Most importantly, if problems occur during application or high loss rates are experienced, consider whether the cause may be on-farm. If in doubt contact Dalton Customer Services, there may be a simple explanation.

Ensure that your applicator is not damaged and is in good working order. It is an important precision tool, treat it with care.

**For assistance or a
free Catalogue or samples call;**

Dalton I.D. Systems Ltd, Dalton House, Newtown Road
Henley-on-Thames, Oxon. RG9 1HG, United Kingdom

**+44 (0) 1491 419 000
or email: info@daltonid.com**